



NEW



Adjustable

**ORMESA®**

• MADE IN ITALY SINCE 1980 •

**Trolli**

More than  
a folding  
system!



Practicality  
at hand



## Advanced Design for a no limits mobility

**Adjustable Trolli** by Ormesa is a compact and light **seating system for teens and adults**, ideal to move in any environment.

The system is able to offer even to people with complex disabilities, a **light seat, easy to manage**, usable in all those transfers for which the use of a classic posture system would become difficult and not easy to implement.

It is easy to handle thanks to the **international patented system** which allows you to **fold it with one single operation**. Once folded, it remains compact, **occupying the minimum space**.

Adjustable Trolli has a **stable and resistant structure** guaranteed by the **double cross** positioned behind the backrest and under the seat which gives greater stability and comfort.

The **upholstery** is **made with advanced technological materials**: it is extremely **breathable, resistant, easily washable** and **fire retardant**. It is available in **two colors**: red and black.

Thanks to a **wide range of available components**, it easily meets every need.



**Adjustable Trolli is CRASH TESTED** according to **ISO 7176/19** and **ANSI RESNA WC/19** standards.

PRODUCT  
MARKED **CE**

ADJUSTABLE TROLLI is a MEDICAL DEVICE COMPLYING WITH THE SAFETY REQUIREMENTS DEFINED BY REGULATION (EU) 2017/745





# Practical and light

## Sea, mountain, city: no problem!



**Adjustable Trolli** has an aluminium **folding frame**, painted with non-toxic epoxy powders.

**Once folded it remains compact:** this allows you to **carry it on its rear wheels like a travel trolley**.

Its **low weight** and its **simplicity of folding**, makes it easy to lift or move: it is perfect if you need to store it in the car or if you need to take public transport.

**Adjustable Trolli** is the **ideal solution** for those looking for **stability, practicality and comfort, all in one!**



# Technical features and adjustments

## Upholstery

The upholstery of Adjustable Trolli has a **honeycomb** leading-edge structure, therefore it is **extremely breathable in contact with the skin**. The structure of the fabric creates an "air chamber" that allows both **air circulation for maximum breathability** and an **effective natural thermal insulation**.

**The double layer is a guarantee of:**

- › **Hygiene:** Thanks to the excellent breathability, it reduces sweating and bacteria formation.
- › **Washing:** despite the upholstery is not removable, the fabric composition allows easy sanitization and washing.
- › **Quick drying.**
- › **Resistance:** High resistance to abrasion. The high resistance texture maintains its performance even at high loads (ISO 12947 test).
- › **Fire retardant:** according to EN 1021-1:2014 and EN 1021-2:2014.







## Postural headrest (mod.863)

Adjustable in tilt, depth and in longitudinal slide thanks to levers and height adjustable by screw. Safe adjustment without the risk of losing the screw.



Foldable for transport. It closes inside the stroller, not increasing the overall dimensions.



**NEW**




## Backrest extension (mod.823)

with slots for 835 and 865 headrest.





Foldable for transport. It closes inside the stroller, not increasing the overall dimensions.

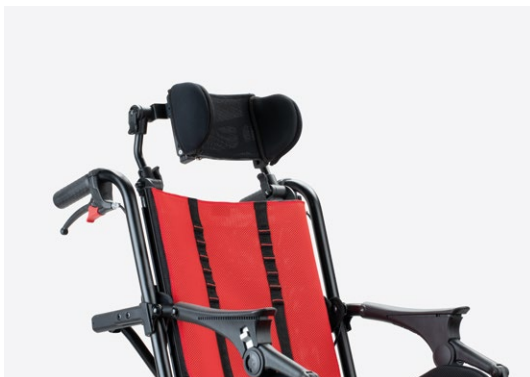

## Backrest

Simple adjustment in height by sliding, without the risk of losing the screw.

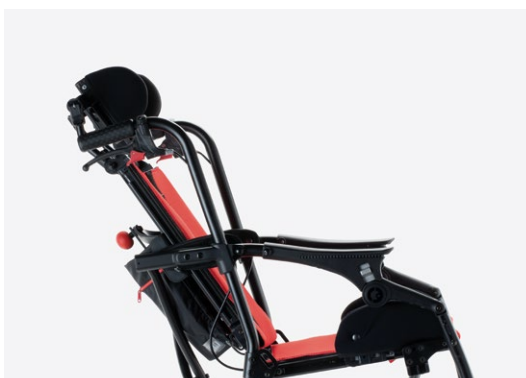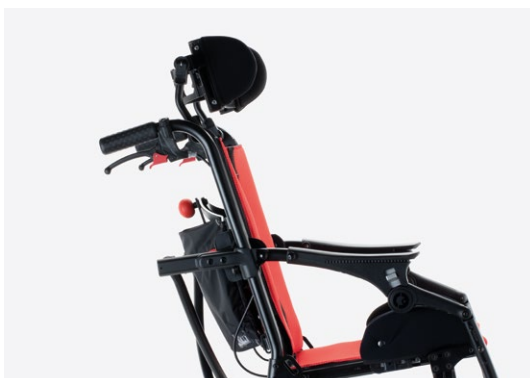

Reclining in three positions.

With a bag on the back to store the manual and the supplied wrench. The back cross ensures stability, comfort and endurance.



## Seat

Ergonomic and slightly tilted seat to prevent slipping of the pelvis and ensure greater comfort. Adjustable in depth by sliding and in width thanks to the additional component 958 padded pelvic side supports. Simple adjustment without the risk of losing the screw. The cross below ensures stability, comfort and endurance.



## Handles

Push handles with manual and parking brakes. Very useful handles to manage the stroller in different environments. Adjustment in height with the component 951.



## Legrest

Adjustable in height.

## Footrests

Tilting and swivel footrest in line with the frame without encumbrance. Adjustment in tilt with the component 956.

The calf support band is also included for more comfort.



## Wheels

With “Ormesa Design” original profile. Solid, light and **all-terrain wheels**.

**Front solid swiveling wheels.** 7” (17 cm) diameter for size 36 and 40; 8” (20 cm) diameter for size 43.

**Rear solid wheels** with **independent drum brakes**, 10” (25 cm) diameter for size 36 and 40; 12” (30 cm) diameter for size 43. **Overcoming step bar** on both rear wheels.



# Components

NEW



**865 Padded headrest** height adjustable. Only with backrest extension.

NEW



**835 Shaped and enveloping headrest** height adjustable. Only with backrest extension.



**838 Trunk side supports** adjustable in height, width and rotation. Available in two sizes.



**868 Wrappable and flexible trunk supports** Multidjustable in height, width and inclination.



**958 Padded pelvic side supports**



**947 Pelvic belt with variable angle** Available in two sizes.



**920 Four point pelvic belt** Available in two sizes.



**853 Vest harness**


# Components



**853 slim Four point shaped harness.**



**968 Padded covers for armrests.**  
Easily removable and washing.

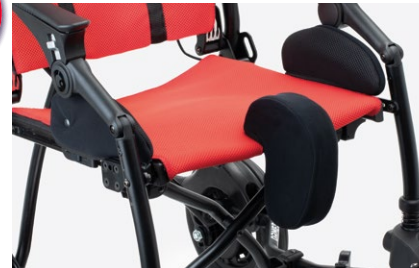

**834N Padded abduction block**  
removable. Available in two sizes.



**839 Front handlebar.**  
Adjustable in tilt. Easy to remove to fold the stroller.



**951 Height adjustable push handles. TOOLS FREE!**  
It increases the total height of max. 7 cm/2,7 in (94-101 cm/37-39,7 in).



**960 Heel rests.**



**956 Footrests adjustable in tilt.**

# Components



**819 Elevating canopy foldable with the frame.**



**966 Kit self propelling wheels (22")** with drum brake, anti-tip device and quick release.  
The additional component padded pelvic side support (958) can't be fixed.



**825 Rain cover**  
to fix on the 819 canopy.



**825B Back and lateral protection cloth**  
it can be easily attached to the 819 canopy by means of zip and snap buttons, for greater protection from wind and rain.



# Components



**812 Front wheel directional locks.**



**858 Shopping basket.**



**844 Padded side protection covers.**



**891 Set of 4 tie down hooks**  
for the forward-facing transport of the  
pushchair in a motor vehicle (private/  
cars, buses..).  
*See User and Maintenance Manual  
provided with the aid.*








**Adjustable Trolls has passed  
the CRASH TEST according to ISO  
7176/19 and ANSI RESNA WC/19**

# Adjustable Trolli sizes and weights

## with backrest extension 823

Unit of Measurement: cm (in); kg (lb)

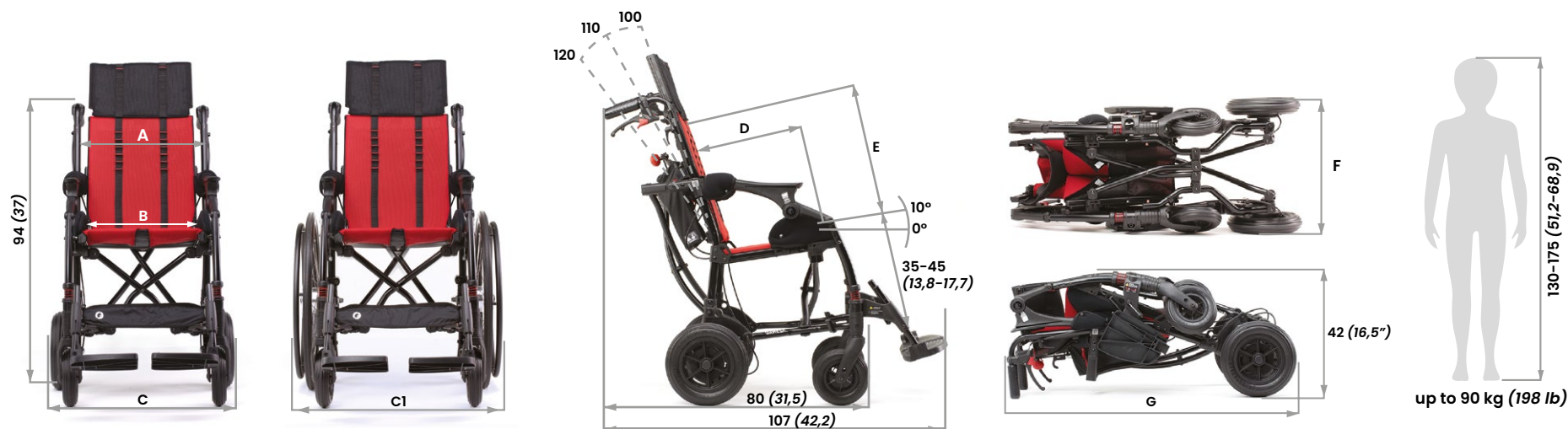

	A	B	C	C1 with 966	D	E	F	G	Tot. weight
size 36	39 (15,4)	36 - 32 - 28 * (14,2 - 12,6 - 11 *)	59 (23,2)	64 (25,2)	36-46 (14,2-18,1)	46-56 (18,1-22)	42 (16,5)	94 (37)	15,1 (33,2)
size 40	43 (16,9)	40 - 36 - 32 * (15,7 - 14,2 - 12,6 *)	63 (24,8)	68 (26,8)	36-46 (14,2-18,1)	46-56 (18,1-22)	46 (18,1)	94 (37)	16,1 (35,4)
size 43	46 (18,1)	43 - 39 - 35 * (16,9 - 15,4 - 13,8 *)	66 (26)	71 (27,9)	36-46 (14,2-18,1)	46-56 (18,1-22)	46 (18,1)	97 (38,1)	17,1 (37,6)


\* using the 958 padded pelvic side supports the width is reduced by 4 cm (1,6 in) or 8 cm (3,2 in) removing the internal padding.  
 36-40-43 cm (14,2-15,7-16,9 in) width without padded pelvic side supports.



# Adjustable Trolli sizes and weights


## with postural headrest 863

Unit of Measurement: cm (in); kg (lb)

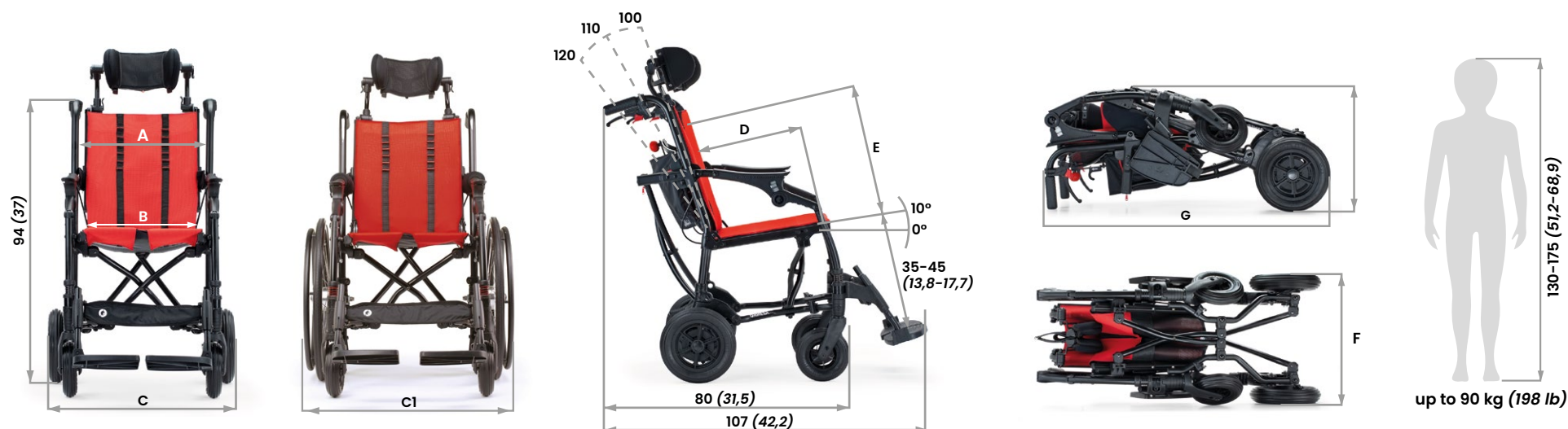

	A	B	C	C1 with 966	D	E	F	G	Tot. weight
size 36	39 (15,4)	36 - 32 - 28 * (14,2 - 12,6 - 11 *)	59 (23,2)	64 (25,2)	36-46 (14,2-18,1)	46-56 (18,1-22)	42 (16,5)	94 (37)	15,1 (33,2)
size 40	43 (16,9)	40 - 36 - 32 * (15,7 - 14,2 - 12,6 *)	63 (24,8)	68 (26,8)	36-46 (14,2-18,1)	46-56 (18,1-22)	46 (18,1)	94 (37)	16,1 (35,4)
size 43	46 (18,1)	43 - 39 - 35 * (16,9 - 15,4 - 13,8 *)	66 (26)	71 (27,9)	36-46 (14,2-18,1)	46-56 (18,1-22)	46 (18,1)	97 (38,1)	17,1 (37,6)

\* using the 958 padded pelvic side supports the width is reduced by 4 cm (1,6 in) or 8 cm (3,2 in) removing the internal padding.  
36-40-43 cm (14,2-15,7-16,9 in) width without padded pelvic side supports.

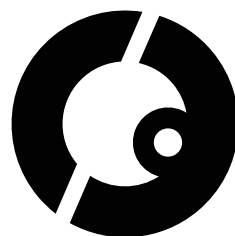

**CE** according to MDR (EU) 2017/745

COMPANY WITH  
QUALITY SYSTEM  
CERTIFIED BY DNV  
[ISO 13485](#)

---

**Ormesa srl** - Via delle Industrie 6/8 | 06034 Foligno (PG) - ITALY  
Tel. +39 0742 22927 | FAX +39 0742 22637 | [info@ormesa.com](mailto:info@ormesa.com)

**[www.ormesa.com](http://www.ormesa.com)**

