



ORMESA®
• MADE IN ITALY SINCE 1980 •

GRILLO chair





A chair for a thousand activities.

Grillo Chair is an **ergonomic** and **adjustable chair** indicated for children with disabilities who need support while performing recreational, educational and daily activities.

It is an **ergonomic seat** designed to provide **maximum comfort** for the child who, by improving sitting posture, is encouraged to develop manual motor skills, attention, and thus social inclusion.

The **load-bearing upholstery, extremely breathable and durable**, allows the child to live a **rewarding and hassle-free sitting experience**. Available in pink and blue.

Thanks to a wide range of postural components, the **Grillo Chair is easily adaptable** to meet the needs of every child. In class and at home the **child will feel safe** and this will help him in various activities and to approach to peers.

PRODUCT
MARKED 

GRILLO CHAIR IS A MEDICAL DEVICE COMPLYING WITH THE SAFETY REQUIREMENTS DEFINED BY REGULATION (EU) 2017/745



Perfect Matching

Stability, posture and practicality



Grillo Chair has a compact aluminum frame, painted with non-toxic epoxy powders.

The **gas-spring height adjustment system** allows the chair height to be changed even with the child seated.

Safety system to avoid unintentional activation.

Grillo Chair is the ideal solution at school and home: it can be combined with any surface for educational and play activities or with multifunctional tables such as **Ormesa's Multifunctional Table**, creating a **perfect match between stability, posture and practicality.**

Smart upholstery

Leading-edge cover

The upholstery of Grillo Chair has a **honeycomb** leading-edge structure, therefore it is **extremely breathable in contact with the skin**. The structure of the fabric creates an “air chamber” that allows both **air circulation, for maximum breathability**, and an effective **natural thermal insulation**.

The double layer is a guarantee of:

- › **Hygiene:** thanks to the excellent breathability, it reduces sweating and bacteria formation.
- › **Washing:** the fabric composition allows easy sanitization and washing, despite the upholstery is not removable.
- › **Quick drying.**
- › **Resistance:** High resistance to abrasion. The high resistance texture maintains its performance even at high loads (ISO 12947 test).
- › **Fire retardant:** according to EN 1021-1:2014 and EN 1021-2:2014.



Technical features and adjustments






Backrest

Reclines with two green levers easily and securely. Adjustment in height by sliding without any risk of losing the screw.



Seat

Ergonomic and slightly tilted seat to prevent the pelvis from slipping ensuring greater comfort. Adjustable in depth by sliding continuously and precisely. Adjustable in width thanks to the optionable component 958 multi-adjustable padded pelvic side supports.

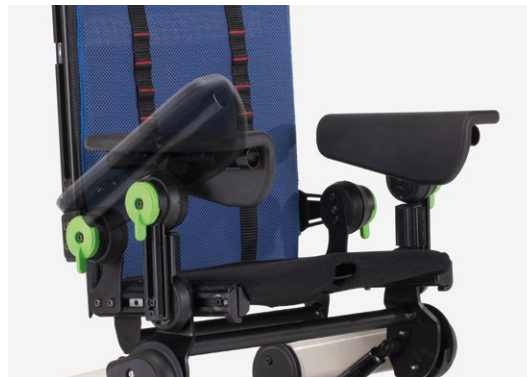

Armrests

soft and adjustable in height and inclination.

Components




947 Pelvic belt with variable angle

Available in three sizes.




920 Four point pelvic belt

Available in two sizes.

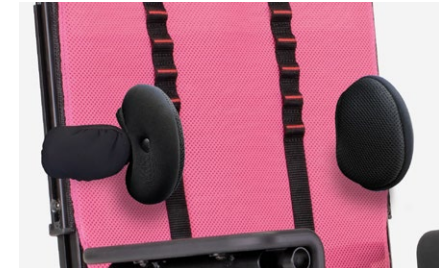


838 Trunk side supports

Adjustable in height, width and rotation.
Available in two sizes.

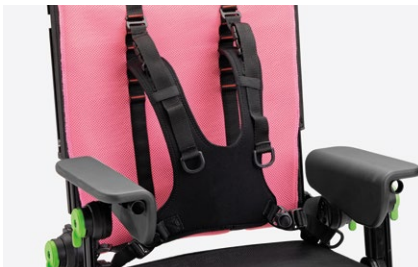


853 Vest harness

Available in two sizes.

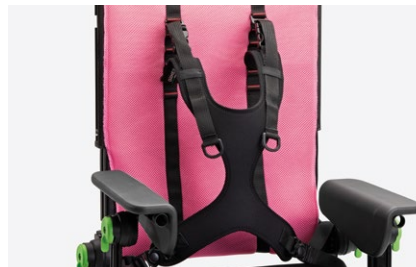

853 slim Four point shaped harness

Only for Grillo Chair size Medium.

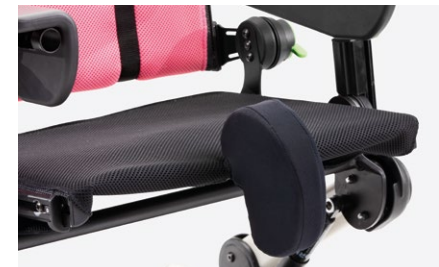

834N Narrow abduction block

Available in two sizes.

Components



958R Padded pelvic side supports

Adjustable in width, rotation and depth.



824 Tray with wrap-around recess Adjustable in depth, height and tilt with armrests.

It is made of resistant and hygiene material, no edge design, it does not retain any residue of food.



823 Backrest extension for the head support.



835 Shaped headrest with lateral and occipital support, adjustable in height. Only with 823 backrest extension.




865 Padded headrest adjustable in height . Only with 823 backrest extension.



959 Set of 4 wheels with brakes as an alternative to the 4 black glides. 75 mm swivel wheels all equipped with brakes. **Supplied with footrest.**

For easy transport and storage. Allows the transfer even with the child seated.

Grillo Chair sizes and weight

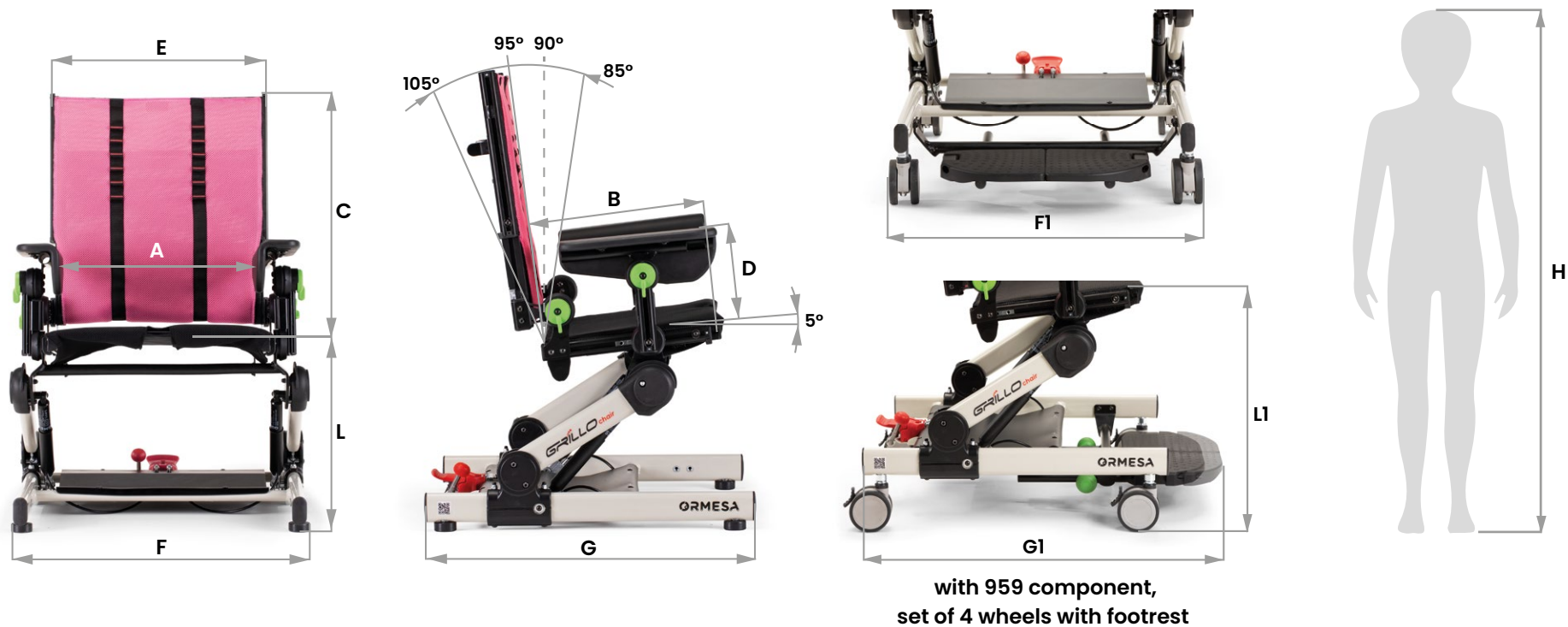


	A*	B	C	D	E	F	F1	G	G1	L	L1	H	Tot. weight	Max. Load
Small	30 cm	20-30 cm	30-45 cm	15-18 cm	36 cm	48 cm	50,5 cm	50 cm	52-64 cm	20-31 cm	28-39 cm	90-120 cm	10,7 kg	35 kg
	11,8 in	7,8-11,8 in	11,8-17,7 in	5,9-7 in	14,1 in	18,8 in	19,8 in	19,6 in	20,4-25,1 in	7,8-12,2 in	11-15,3 in	35,4-47,2 in	23,5 lb	77,1 lb
Medium	35 cm	28-43 cm	35-55 cm	16-20 cm	41 cm	52 cm	54,5 cm	56,5 cm	62-73 cm	25-40 cm	33-48 cm	110-145 cm	13 kg	45 kg
	13,7 in	11-16,9 in	13,7-21,6 in	6,2-7,8 in	16,1 in	20,4 in	21,4 in	22,2 in	24,4-28,7 in	9,8-15,7 in	12,9-18,8 in	43,3-57 in	28,6 lb	99,2 lb

* with component 958R multi-adjustable pelvic side supports: size SMALL 20-30 cm (7,8-11,8 in); size MEDIUM 25-35 cm (9,8-13,7 in)

A man with a beard and a ponytail, wearing a blue shirt and jeans, is kneeling on the floor of a classroom. He is assisting a young boy with a tablet computer. The boy is sitting in a specialized, adjustable chair with a black frame and blue mesh backrest, which has 'ORMESA' written on the side. The boy is wearing a white t-shirt with a graphic and is looking at the tablet. The classroom has orange walls, a sink, and various toys on a table in the background.

We thank Sterpete's Preschool
for their support and hospitality.

Giulio and his story
await you in the Ormesa
official website, together
with other protagonists
of our fantastic global
community.

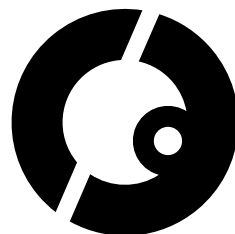

A special thanks to the families who participated with enthusiasm in our project. It is thanks to our customers and their stories that we find the energy and stimuli to evolve day by day.

CE complies with MDR (EU) 2017/745

COMPANY WITH
QUALITY SYSTEM
CERTIFIED BY DNV
[ISO 13485](#)

Ormesa srl - Via delle Industrie 6/8 | 06034 Foligno (PG) - ITALY
Tel. +39 0742 22927 | FAX +39 0742 22637 | info@ormesa.com

www.ormesa.com

